

How to transmit your easypod[®] data




- Open the lid of the transmitter.
- The left status light may start flashing yellow slowly (alternating between 1 second on, 1 second off), as the transmitter searches for a signal.




- The first connection to a mobile network may take up to 120 seconds (2 minutes). The second and all future times, the connection will take less than 30 seconds.
- When the transmitter has successfully connected to the network, the left status light will flash green slowly (alternating between 1 second on, 1 second off).

○ **NOTE:** If you leave the lid open for more than 5 minutes, the transmitter will go into stand-by and will not activate when you place the easypod[®] auto-injector in the cradle. Close and re-open the lid to re-start the transmitter.

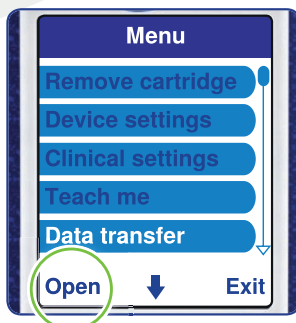

- Take your easypod out of the refrigerator and power it on.
- Select '**Data transfer**' from the easypod® main menu and press '**Open**'.
- The easypod® screen will display the '**Place in transmitter**' message.
- Press the '**Yes**' button on your easypod® auto-injector.
- Place your easypod® into the transmitter cradle with the screen facing you.

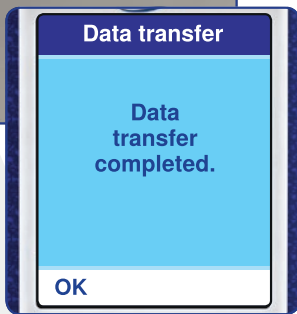

- The transfer of information will start automatically.
- The left status light will flash green quickly (4 flashes per second) to indicate that information is being transferred.
- When the data transfer is successfully completed (after approximately 1 minute), the left status light will stop flashing and both lights will turn green.
- 'Data transfer completed' will appear on the easypod® screen.
- Press the 'OK' button.

○ **NOTE:** Do not remove your easypod® from the transmitter or disconnect the power cable while data is being transferred. If this happens you should close the lid and restart the procedure.



- Remove your easypod® from the transmitter and close the lid.
- You have successfully transferred your information.
- Turn off your easypod®, put it back in its box and return it to the refrigerator in vertical position for safe storage.
- Unplug the transmitter and store it in a secure location. Do not store your transmitter in the refrigerator.

IMPORTANT: Do not store your easypod® in your transmitter. Your easypod® containing a cartridge of Saizen® should be stored in the refrigerator at 2-8°C (36-46°F).

MERCK

This leaflet has been created by Merck and it is intended for patients who have been prescribed Saizen® (Somatropin)
UI-SAI-00157, May 2025